



---

## Clearlight HALO ONE® salt therapy device



### Price

[Ask a question about this product](#)

**Description** Incorporate salt therapy into your daily wellness routine!

Salt therapy is a holistic treatment that replicates the natural microclimate of a salt cave by dispersing a high concentration of microsalt particles into the air.

The HALO ONE® device has been designed and manufactured to provide beneficial salt therapy sessions in wellness areas, spas, treatment centers, in your home, bathroom or office, but also inside infrared saunas.

[Learn more about the benefits of salt therapy for skin, rhinitis and respiratory problems...](#)

---



### **Production of the smallest salt particles with Vibrating Mesh technology**

This quiet, easy-to-use, award-winning device uses a 3% pharmaceutical-grade saline solution. With patent-pending Vibrating Mesh technology, the solution is evaporated and then forced through a vibrating mesh to create salt particles between 1-10 microns in size.

The tiny salt particles are then dispersed into the air for you to breathe.

The HALO ONE® device is the only salt therapy generator that can produce such small salt particles.

This means that the salt will reach a larger area of your lungs than any other salt therapy device on the market, helping you breathe easier.

### **Pharmaceutical grade salt**

The Halo One device uses 99.99% pure sodium chloride (pharmaceutical grade salt).

It does not contain fillers or caking agents.

It comes from a natural source and undergoes natural processing to remove all impurities.

It is the same salt that hospitals use and the same type of salt used in salt therapy research.

### **Portability and Ease of Use**

The HALO ONE device operates with a long-lasting rechargeable battery and is lightweight, allowing for easy use in any small enclosed space, even inside an infrared sauna.

### **What is the difference between Halo One and using saline nasal washes?**

Liquid salt therapy involves the use of a salt solution in water, such as saline irrigation, sprays, or inhalation of salt vapor.

Although these methods can provide temporary relief by clearing the upper respiratory tract, their effectiveness is limited.

Salt particles, being liquid, are larger and heavier, so they cannot penetrate deep into the lungs and nasal cavities, where toxins and germs accumulate.

In contrast, dry halotherapy produces extremely fine salt particles (1-5 micrometers) that can penetrate deep into the respiratory system, reaching the alveoli of the lungs.

These tiny particles have powerful anti-inflammatory and antibacterial properties that help remove toxins and reduce inflammation throughout the respiratory system.

Although the HALO ONE device uses a saline solution (water + salt), it uses a specialized vibrating mesh technology to produce micron-sized salt particles (1-10 microns) that are dispersed into the air and are essentially dry when inhaled (fine salt aerosols), so the end result is the same as traditional dry salt therapy (i.e., inhalation of dry salt particles).



---

### **Suitable for use in small enclosed spaces**

To achieve a therapeutic salt concentration in the space, we recommend using the device in rooms up to 9.5 m<sup>2</sup> in area.

The HALO ONE is not designed for high humidity areas such as swimming pools or steam rooms. Use in these conditions could affect performance and longevity. High humidity levels may encourage the growth of biological organisms in the environment.

### **Also suitable for use in an infrared sauna**

The HALO ONE® can be used in an infrared sauna up to a maximum temperature of 60°C for no more than one continuous hour.

The HALO ONE® unit is not designed for continuous use in the sauna and should be removed after each use if the sauna is used for multiple sessions per day.

Caution: the HALO ONE® device cannot be used simultaneously with the Red Light Therapy (RLT) phototherapy tower as it is sensitive to salt.

### **Long battery life**

There's no need to install or replace batteries – the unit uses only an internal rechargeable battery that is charged with the USB cable and adapter.

Battery life when fully charged is ~700 minutes (full charge to full discharge).

Full recharge time is ~4 hours and 40 minutes (0% to 100%).

You can use the device while it is plugged into the outlet.

The unit turns off after 1 minute of inactivity when unplugged to save battery power.

### **Award-winning design**

HALO ONE® Salt Therapy is distinguished by its smart and ergonomic design, a fact that has been recognized with international awards in the field of design.

The Halo One device has been honored with a GOOD DESIGN AWARD, an international institution for industrial design.



# GOOD DESIGN AWARD

### **Technical characteristics**

External Dimensions: L 25cm x W 33cm x 10 cm

Weight: 1.4 kg

Power Sources: 1. Dedicated USB PD 45W power adapter (Power Supply 100~240V AC 50/60Hz,

Output: 15V 3A) 2. Internal 5000mAh lithium battery

Operating Temperature: 10°C ~ 40°C

Power Consumption: ~ 35W

Main Unit Material: PC/ABS (Polycarbonate/Acrylonitrile Butadiene Styrene - strong, impact-resistant plastic blend)

Accessories: Wall Mount, Power Adapter, USB-C Cable, Screws, User Manual

Warranty: 2 years.

For detailed information on usage, cleaning and maintenance routines, warranty, and best operating practices, download the [Halo One Manual](#) and [Commercial Use Guide](#) .



---

## Price and Purchase Process

[For the current product price, download the latest price list by clicking here.](#)

The Halo One device includes 30 salt capsules (each 20 mL salt capsule is intended for a typical 20-minute salt therapy session).

Additional Salt capsules – pack of 30: €77 +VAT

Additional Salt capsules – pack of 120: €273 +VAT

If you would like to proceed with a purchase, please send us an email at [This email address](mailto:info@home-wellness.com) is being protected from spambots. You need JavaScript enabled to view it.

```
document.getElementById('cloak08e3cad6c7bb243a0bc801d3d162b41a').innerHTML = ''; var prefix = 'ma' + 'il' + 'to'; var path = 'hr' + 'ef' + '='; var addy08e3cad6c7bb243a0bc801d3d162b41a = 'info' + '@'; addy08e3cad6c7bb243a0bc801d3d162b41a = addy08e3cad6c7bb243a0bc801d3d162b41a + 'home-wellness' + '.' + 'gr'; var addy_text08e3cad6c7bb243a0bc801d3d162b41a = 'info' + '@' + 'home-wellness' + '.' + 'gr'; document.getElementById('cloak08e3cad6c7bb243a0bc801d3d162b41a').innerHTML += '+' + addy_text08e3cad6c7bb243a0bc801d3d162b41a + '+'; with:
```

1. The exact model and/or additional Clearlight accessories you wish to order
2. Shipping details (name, address, mobile phone number)
3. Invoicing details, in case an invoice is issued instead of a receipt
4. If the purchase concerns domestic or commercial use (spa, gyms, treatment rooms, etc.)

The sale of this product is made exclusively through a mediation process by Home Wellness and the sales contract is concluded directly with Clearlight Saunas Europe GmbH ([terms](#)). All payments, invoicing, warranty and returns are processed directly with Clearlight.

[Learn more about the benefits of salt therapy for skin, rhinitis and respiratory problems...](#)

## Simple way to use

Charge the device by opening the rubber cover covering the charging port and connecting the USB-C cable to the adapter and the power outlet.

The unit is charging when the battery level indicator is flashing. When the unit is fully charged, the charging indicator will turn off and the battery level indicator will stop flashing.

Place a saline capsule in the special port.

To turn on HALO ONE®, press and hold the POWER button for 2 seconds.

Press the POWER button again and the unit will begin a 20-minute salt therapy session (with remaining time displayed).

Once the capsule is finished, the device will stop automatically.

## HALO ONE replacement capsules

HALO ONE refill capsules contain 20 mL of 3% medical grade saline each, sealed and ready to use.

Each capsule provides the optimal salt to water ratio, pre-measured by the manufacturer. Use only saline capsules supplied by HALO ONE®. Use of other solutions may adversely affect functionality and cause damage to the device.

Each 20 mL saline capsule is intended for a typical 20-minute salt therapy session.

The device is calibrated so that a full capsule is aerosolized in approximately 20 minutes, matching the typical duration of a dry salt therapy treatment. After the session, the capsule is emptied and discarded.

If you try to reuse a partially used capsule later, the session will not start (the status indicator will flash blue and the empty container indicator will light up) because the salt concentration will no longer be optimal to start the treatment.

---



---

**Optional accessory: HALO ONE Breathe Easy Face Mask**

The Clearlight ONE HALO ONE Breathe Easy Salt Therapy Face Mask is an accessory designed to enhance your salt therapy experience, especially for respiratory problems and when it is not desirable for other people to use salt therapy or the space is large enough for salt therapy use (>9.5m<sup>2</sup>).

By directing more micronized saline directly into your lungs, this comfortable and easy-to-use face mask increases salt intake and eliminates the need for a confined space to use the HALO ONE device.

Ideal for people with asthma, allergies, COPD and other respiratory problems, the HALO ONE Breathe Easy Face Mask offers a more focused and effective salt therapy session.

Its price is €51+VAT. In case of issuing an invoice, VAT is not charged.

**Recommended frequency of use**

Salt therapy offers the best results when applied regularly.

The benefits are cumulative, so consistency and frequency are key.

---



---

People with chronic problems will greatly benefit from using HALO ONE at least 3-4 times a week.

Those looking to incorporate salt therapy into their regular wellness routine will benefit from even a weekly session.

#### **Why is HALO ONE® set for 20 minutes of use?**

Inhaled dry salt particles work best when they have time to reach both the upper and lower respiratory tract.

The first 5-10 minutes mainly cover the upper airways (nose, throat).

The next 10 minutes allow the finer particles (1-5 µm) to travel deeper into the bronchioles and alveoli.

After ~20 minutes, your mucous membranes are already saturated with salt and the benefit curve flattens out.

In the final minutes, the density of the aerosol naturally decreases because there is less liquid left in the capsule. This acts as a “soft landing” for your airways – avoiding a sudden transition from high particle concentration to zero.

#### **Can I use it continuously for more than 20 minutes?**

We recommend that you do not extend sessions to > 20 minutes (e.g. by adding another capsule) and that you leave at least 6-12 hours between sessions to avoid over-drying your airways.

More salt isn't always better – too much can be irritating rather than healing.

#### **Maintenance instructions**

- With normal use, HALO ONE® should not cause problems in the room environment.
- However, fine salt particles can have a corrosive effect on various materials, such as metal, concrete and electronic devices. These objects may be damaged as a result and should not be left near the HALO ONE® during operation.
- We recommend that care be taken to cover, seal and/or remove any electronic equipment or objects that could be affected by fine salt particles and that surfaces in the room where the unit is operating are regularly cleaned and vacuumed to remove any salt dust.
- With the fan button you can increase the production of salt particles. If humidity occurs in the room, reduce the power of the humidifier. Do not let absorbent materials, such as carpets, curtains, drapes or tablecloths, get wet.
- Remove the device from the infrared sauna when not in use to prevent unnecessary heat exposure.
- Do not leave water in the tank when the Halo One is not in use.
- After each use, clean the unit with the special brush to maintain its performance and longevity. Avoid touching the center of the mesh, as it can be easily damaged.
- It is recommended to clean the water tank every three days. Use a clean soft cloth with neutral detergent or lukewarm water to clean the capsule holder opening.
- Do not immerse the unit in water. Do not touch or press the metal outlet nozzle with excessive force.
- If you are not going to use the unit for a long time, unplug the power adapter and charging cable if connected, remove the saline capsule case and empty all saline remaining in the unit, leave the unit open overnight to dry the internal parts, and then store the unit in a place free from high temperatures or humidity.

#### **Indications**

Respiratory diseases: Asthma, Chronic Obstructive Pulmonary Disease (COPD), Allergic Rhinitis, Sinusitis, Chronic Cough, Frequent Colds, Pharyngitis, Tonsillitis, Pneumonia, Cystic Fibrosis  
Skin diseases: Eczema, Dermatitis, Acne, Rosacea, Onychomycosis, Skin aging, Psoriasis in the maintenance phase, Sebaceous gland hyperfunction (sebaceous seborrhea) – pyodermatitis, Thermal skin lesions, Postoperative conditions (cosmetic surgery), Cellulite, Pale skin

[Learn more about the benefits of salt therapy...](#)

#### **Precautions – Contraindications**

Salt therapy is a gentle and safe treatment.

There are usually no side effects other than a salty taste and possibly thirst or a scratchy throat.

Increased coughing and abundant secretions are considered therapeutic because they lead to clearing of the respiratory tract.

Cases of skin irritation are rare and usually resolve within 2-5 sessions. The same applies to

---



---

cases of conjunctivitis, due to irritation of the mucous membrane (it is recommended to keep your eyes closed when in the salt room).

However, if someone is taking any medication or suffering from a serious medical problem, as well as during pregnancy, please seek medical advice before any treatment. Also, children are recommended to use it with adult supervision.

Not recommended for people with fever, open wounds and serious heart/kidney diseases without medical advice, in severe cases of chronic lung disease, hyperthyroidism, active tuberculosis, acute respiratory disease, high-grade hypertension, neurological disorders, epilepsy, brain dysfunctions, neurological infections and brain trauma, serious infections, acute hematological disorders, infectious diseases, malignant diseases, respiratory system cancer and people with severe depression, neurosis, claustrophobia.

### **Bring the Benefits of the Salt Cave to Your Home!**

You no longer need to visit specialized wellness centers to enjoy the benefits of salt therapy.

With the Halo One Salt Therapy Device, you transform your personal space into a private health sanctuary.

It is ideal for use inside an infrared sauna, combining the benefits of heat with the power of salt, but also in any small space in your home.

Invest in your health. Breathe freely. Feel better.

Don't wait any longer. Give yourself and your family the gift of wellness.

Get your own salt therapy device today!

Ideal for wellness centers, spas, and treatment centers!

---