

HALO ONE

by Clearlight®

Commercial Use Cleaning and Maintenance Manual

This manual supplements the standard user manual with additional guidance specific to commercial environments, where the Halo unit experiences higher frequency and intensity of use.

1. Intended Use

This HALO ONE® unit is intended for dry salt inhalation solely and strictly in accordance with this Manual including (without limitation) only using HALO ONE® salt capsules provided by the manufacturer.

The HALO ONE® can also be used inside a sauna up to a maximum temperature of 140°F (60°C) for no more than one consecutive hour. The HALO ONE® unit is not designed for continuous use in the sauna and should be removed after each use if the sauna is used for multiple sessions in a day.

The HALO ONE® unit will provide a fine salt mist in any room or sauna where it is operated. Care should be taken to cover, seal and/or remove any electronic equipment or objects that could be affected by fine salt particles. Please let common sense guide you when choosing which rooms to use the HALO ONE®. With normal use, the HALO ONE® should not cause any problems to the room environment, but manufacturer / supplier assumes no liability or responsibility for any claims or damages that may arise except as expressly set forth in the HALO ONE® warranty.

2. Cleaning Routine

To maintain optimal performance and hygiene in commercial settings, the Halo unit must be cleaned after each use according to the detailed steps outlined in Section 9 of the user manual, including:

- Brushing the emitter and air inlet using the provided cleaning brush. **Avoid touching the mesh center of the emitter with the brush or cloth, as it can easily be damaged.**
- Removing and draining the saline capsule and water reservoir.
- Rinsing and thoroughly drying the water reservoir.

In addition to these steps, all external surfaces, including metal components (such as sauna heaters) exposed to saline mist or condensation, should be wiped down daily with a soft cloth dampened with mild detergent or neutral cleaner. This helps prevent salt buildup and corrosion, which is not covered by the product warranty.

Important: Regular tap water could lead to damage or discoloration of components. Using distilled water to clean out the water reservoir will prevent scale build up and corrosion.

Tip: Use compressed air to clear any remaining salt residue from the diffuser.

3. Daily Deep Clean

At the end of each day, complete a deep clean following the vinegar solution procedure described in Section 9:

- Prepare a cleaning solution of 20mL distilled water with 10 drops pure white vinegar.
- Fill the reservoir and run the unit for 3-5 minutes to disperse the solution internally.
- Drain, rinse thoroughly with distilled water, and allow to air dry completely.

Regular deep cleaning is essential in commercial use to prevent microbial growth and maintain air quality.

Tip: The easiest way to do this is to take a used capsule, fill it with the mix of distilled water and vinegar, place it back into the machine, and run a cycle.

4. Environmental and Maintenance Precautions

- Ensure the surrounding environment is kept dry and well ventilated to minimise humidity buildup that encourages microbial growth.
- Never allow water to remain in the reservoir when the unit is not in use.
- Avoid direct water spray on electronic components and never touch the metal mist outlet parts directly.
- Damp materials near the unit, such as curtains or tablecloths, should be monitored and kept dry.
- Remove the HALO unit from the infrared sauna when not in use to preserve battery life and prevent unnecessary exposure to residual heat.

5. Use Restrictions - RLT Upgrade

The cooling fan located on the optional Red Light Therapy (RLT) upgrade unit can draw saline mist and salt particles into its electronics if the Halo unit is operated simultaneously. To avoid potential damage (not covered under warranty) the Halo unit must not be used at the same time as the RLT device.

Tip: Protecting the top of the RLT with a cloth while using the Halo One can help reduce salt buildup and lessen the cleaning required on the RLT after Halo use.

**Small rituals.
Big difference.**



Contact Information:

Please visit <https://support.clearlightsaunas.co> for online troubleshooting information.